



replacement parts industries, inc.

"The Alternate Source"™

CALL 800-221-9723 or 818-882-8611 • FAX 818-882-7028
E-MAIL: techsupport@rpiparts.com • WEBSITE: www.rpiparts.com

AIK160 PAWL LATCH KIT INSTALLATION INSTRUCTIONS

1. Refer to illustration below for proper assembly order and assemble as shown.
2. Align hole in Knob and hole in Pawl and install Spring Pin. Place a piece of cloth (or any protective material) between pliers and Knob. Press Spring Pin into Knob until the Pin is flush or below the surface of the Knob.

***Note:** When installing this Kit on an Isolette® C400/C450 QT® Access Panel without a gasket on top edge, on an Isolette® C500/C500 QT® Model XL, or on an Isolette® C2000, you must use the two Spacers included in this Kit. Only One Spacer is needed for other models.

